

Јануари 2024

Известување за безбедност на терен
Perceval Plus аортна срцева валвула без шиене

За внимание на: надлежни органи за вигиланца и корисници на Perceval Plus аортна срцева валвула без шиене вклучени во управувањето со залихите и имплантација на моделите наведени во приложениот Прилог А.

Контакт податоци на носителот на решение за запишување во регистрот на медицински средства достапни во Република Северна Македонија
ДТПУ ПРОМЕДИКА ДОО – Скопје ул. Св. Кирил и Методиј бр.50, тел.+389 2 324 800 и 071/211934, е-пошта: mail:marcikik_m@promedika.com.mk

Причина: Можно присуство на честички во готовиот производ.

Почитувани здравствени работници,

Го добивате ова писмо затоа што, според нашата евиденција, аортните срцеви валвули без шиене Perceval Plus наведени во Прилог А (Формулар за враќање) се испратени до Вашата установа. Овие медицински средства се дел од корективна мерка за безбедност на терен издадена од Corcym S.r.l. Остатокот од ова известување деталзира што ја поттикнува оваа акција, што ќе треба да се направи со засегнатите производи и контактите на Corcym кои можат да пружат помош.



Известување за безбедност на терен (FSN)
Perceval Plus аортна срцева валвула без шиене
Ризик упатен со известувањето за безбедност на терен

1. Информации за засегнатите медицински средства									
1.	1 Тип(ови) на медицинско средство PERCEVAL PLUS е биопротетска валвула со уникатна карактеристика што овозможува позиционирање без шиене и закотвување на местото на имплантот. Изборот на материјали и конфигурација обезбедува биокомпатибилност и хемокомпатибилност на уредот. Протезата PERCEVAL PLUS се состои од ткивна компонента направена од говедски перикардиум стабилизирани во пуфериран раствор на глутаралдехид и самопроширлив стент Nitinol, кој има двојна улога да ја поддржува валвулата и да го фиксира на своето место. Говедскиот перикардиум е подложен на третман за намалување на фосфолипидите. Протезата се стерилизира со раствор на база на глутаралдехид и потоа се третира за неутрализирање на слободните остатоци од алдехид. Срцевата валвула на ткивото PERCEVAL PLUS се испорачува немонтирана. Пред имплантација, дијаметарот на протезата се намалува на соодветна големина за да се вчита на држачот. Валвулата потоа се поставува и се ослободува во коренот на аортата, каде што дизајнот на стентот и неговата способност да примени радијална сила на анулусот овозможува стабилно закотвување на уредот. Потоа, протезата конечно се пакува со помош на асептичен процес на трансфер и се складира во пуферски раствор без алдехиди.								
1.	2. Комерцијално име(иња) Perceval Plus аортна срцева валвула без шиене								
1.	3. Единствени идентификатори на медицинското средство (UDI-DI) <table border="1"> <tbody> <tr> <td>08022057015365</td><td>PVF-S</td></tr> <tr> <td>08022057015372</td><td>PVF-M</td></tr> <tr> <td>08022057015389</td><td>PVF-L</td></tr> <tr> <td>08022057015396</td><td>PVF-XL</td></tr> </tbody> </table>	08022057015365	PVF-S	08022057015372	PVF-M	08022057015389	PVF-L	08022057015396	PVF-XL
08022057015365	PVF-S								
08022057015372	PVF-M								
08022057015389	PVF-L								
08022057015396	PVF-XL								
1.	4. Примарна клиничка намена на медицинското(ите) средство(а) Протезата PERCEVAL PLUS е наменета да ја замени оштетената аортна срцева валвула или нефункционалната аортна протеза преку операција на отворено срце.								
1.	5. Модел на медицинско средство/Каталог/број(еви) на дел PVF-S, PVF-M, PVF-L, PVF-XL								
1.	6. Засегнат опсег на сериски или лот броеви Засегнатите сериски броеви се наведени во Прилог А								



2. Причина за безбедносната корективна мерка на терен (FSCA)	
2.	1. Опис на проблемот со производот Corcym открил дека некои полимерни честички, кои потекнуваат од една од опремата што се користи за терминалната стерилизација, може да бидат присутни во готовите производи наведени во Прилог А. Овие честички се покажале како целосно биокompatibilни и во огромно мнозинство <100 микрони во големина.
2.	2. Ризик што доведува до FSCA Ако честичките, кои може да се најдат на валвуката, влезат во телото, постои ризик тие да мигрираат во непосредната постоперативна фаза. Врз основа на големината на честичките и различните активности за ублажување кои вообичаено се усвојуваат пред и за време на процедурата за имплантација, како што е поставувањето на валвуката во сад со солен раствор пред имплантот, Corcym проценил дека ризикот од повреда на пациентите имплантирани со засегнатото медицинско средство е низок. Сепак, Corcym конзервативно одлучи да ги повлече сите неискористени медицински средства.

3. Вид на дејство за ублажување на ризикот	
3.	1. Дејство што треба да го преземе корисникот ☒ Идентификувајте го медицинското средство ☒ Медицинско средство за карантин ☒ Медицинско средство за враќање Corcym го координира отстранувањето на сите потенцијално засегнати валвули Perceval Plus во вашиот инвентар. Ве молиме погрижете се навремено да ги извршите следниве активности: - користејќи ја листата дадена во Прилог А, проверете го вашиот инвентар за потенцијално засегнати медицински средства испорачани во вашата установа и сè уште се на полиците. - Сегрегирајте и ставете ги во карантин погодените валвули идентификувани според точка 1. - Пополнете го и вратете го Прилогот А (Формулар за враќање) по е-пошта на marcikik_m@promedika.com.mk за да го започнете процесот на враќање на засегнатите валвули Perceval Plus. Вашиот претставник на Corcym, носителот на решение за запишување во регистрот на медицински средства достапни во Република Северна Македонија, ДТПУ Промедика ДОО-Скопје ќе Ве контактира за да го координира враќањето на засегнатите уреди во Corcym S.r.l.
3.	2. До кога треба да се заврши акцијата? Активностите наведени во точките 1, 2, 3 погоре ќе бидат завршени до 8 јануари 2024 година.

3.	3. Дали е потребен одговор од клиентот? (Ако одговорот е да, приложена форма во која е наведен рокот за враќање)	Да
3.	4. Дејство што го презема производителот ☒ Отстранување на производот Вашиот претставник на Corcym носителот на решение за запишување во регистрот на медицински средства достапни во Република Северна Македонија, ДТПУ Промедика ДОО - Скопје ќе ве контактира за да го координира враќањето на засегнатите медицински средства во Corcym S.r.l.	
3.	5. До кога треба да се заврши акцијата?	1-ви Март 2024
3.	6. Дали FSN се бара да му се пренесе на пациентот /корисникот?	Не

4. Генерални информации		
4.	1. Вид на FSN	Нова
4.	2. Дополнителни совети или информации кои веќе се очекуваат во последователниот FSN?	Нема
4.	3. Информации за производителот (За детали за контакт на локалниот претставник погледнете ја страницата 1 од овој FSN)	
	а. Име на производител	Corcym S.r.l.
	б. Адреса	Strada Crescentino, 13040 Saluggia (VC) - Италија
	в. Адреса на веб-страница	www.corcym.com
4.	4. Надлежниот (Регулаторниот) орган на вашата земја е информиран за оваа комуникација до клиентите.	
4.	5. Список на прилози/ додатоци:	Прилог А (Образец за враќање)
4.	6. Име/ Потпис	Марија Марцикиќ Одговорно лице за материовигиланца



	Пренесување на ова Известување за безбедност на терен
	Ова известување треба да се пренесе до сите оние кои треба да бидат свесни во вашата установа или до која било установа каде што се пренесени потенцијално засгнатите медицински средства. Ве молиме проследете го ова известување на други установи на кои има влијание оваа безбедносна корективна мерка. Ве молиме бидете свесни за ова известување и дејството што резултира во соодветен период за да се обезбеди ефективност на корективните мерки. Ве молиме пријавете ги сите инциденти поврзани со медицинското средство до претставникот на Corcym во Република Северна Македонија, носителот на решение за запишување во регистрот на медицински средства достапни во Република Северна Македонија ДПТУ Промедика ДОО - Скопје кај Марија Марцикиќ на тел. 071211934 и е-пошта: marcikik_m@promedika.com.mk и до националниот надлежен орган, Агенцијата за лекови и медицински средства (ул. Св. Кирил и Методиј бр.54 кат 1) или по електронски пат преку веб-страницата на Агенцијата http://malmed.gov.mk/ бидејќи ова обезбедува важна повратна информација.



FSCA REC-000384 ДОДАТОК 1
ЛИСТА НА ЗАСЕГНАТИ СЕЕРИСКИ
БРОЕВИ

SERIAL NUMBER	ITEM	COUNTRY
PE6661C	PVF-S	AUSTRIA
PE6662C	PVF-S	AUSTRIA
PE7022C	PVF-M	AUSTRIA
PE7091A	PVF-M	AUSTRIA
PE7125C	PVF-L	AUSTRIA
PE7132C	PVF-L	AUSTRIA
PE7152C	PVF-L	AUSTRIA
PE7161C	PVF-M	AUSTRIA
PE7163C	PVF-M	AUSTRIA
PE7202C	PVF-M	AUSTRIA
PE7204C	PVF-M	AUSTRIA
PE7237C	PVF-L	AUSTRIA
PE7269C	PVF-L	AUSTRIA
PE7270C	PVF-L	AUSTRIA
PE7271C	PVF-M	AUSTRIA
PE7276C	PVF-L	AUSTRIA
PE7277C	PVF-L	AUSTRIA
PE7278C	PVF-L	AUSTRIA
PE7286C	PVF-M	AUSTRIA
PE7287C	PVF-M	AUSTRIA
PE7291C	PVF-M	AUSTRIA
PE7336C	PVF-M	AUSTRIA
PE7345C	PVF-XL	AUSTRIA
PE7354C	PVF-XL	AUSTRIA
PE7355C	PVF-XL	AUSTRIA
PE7360C	PVF-XL	AUSTRIA
PE8539A	PVF-S	AUSTRIA
PE8556A	PVF-S	AUSTRIA
PE8559A	PVF-S	AUSTRIA
PE8560A	PVF-S	AUSTRIA
PE9972A	PVF-S	AUSTRIA
PF0272A	PVF-S	AUSTRIA
PF0339A	PVF-S	AUSTRIA
PF2295A	PVF-M	AUSTRIA
PE6376A	PVF-XL	BELGIUM
PE6482A	PVF-XL	BELGIUM



PE6483A	PVF-XL	BELGIUM
PE6485A	PVF-XL	BELGIUM
PE6678C	PVF-L	BELGIUM
PE6869C	PVF-M	BELGIUM
PE6893C	PVF-S	BELGIUM
PE6899C	PVF-L	BELGIUM
PE6900C	PVF-L	BELGIUM
PE6902C	PVF-S	BELGIUM
PE6907C	PVF-L	BELGIUM
PE6910C	PVF-L	BELGIUM
PE6915A	PVF-M	BELGIUM
PE6942A	PVF-L	BELGIUM
PE6946A	PVF-L	BELGIUM
PE6963A	PVF-S	BELGIUM
PE6984A	PVF-S	BELGIUM
PE7021A	PVF-M	BELGIUM
PE7080C	PVF-S	BELGIUM
PE7095A	PVF-M	BELGIUM
PE7103A	PVF-M	BELGIUM
PE7151C	PVF-L	BELGIUM
PE7279C	PVF-L	BELGIUM
PE7305A	PVF-XL	BELGIUM
PE7343C	PVF-XL	BELGIUM
PE7429C	PVF-M	BELGIUM
PE7468A	PVF-S	BELGIUM
PE7524C	PVF-XL	BELGIUM
PE7557A	PVF-S	BELGIUM
PE9844A	PVF-S	BELGIUM
PE9944A	PVF-M	BELGIUM
PE9948A	PVF-M	BELGIUM
PF0431A	PVF-L	BELGIUM
PF0451C	PVF-S	BELGIUM
PF1372A	PVF-XL	BELGIUM
PF1429A	PVF-XL	BELGIUM
PF1778A	PVF-S	BELGIUM
PF1921A	PVF-XL	BELGIUM
PF1923A	PVF-XL	BELGIUM
PF2072A	PVF-XL	BELGIUM
PF2180A	PVF-XL	BELGIUM
PF2190A	PVF-XL	BELGIUM
PF2191A	PVF-XL	BELGIUM
PF2242A	PVF-XL	BELGIUM
PF2243A	PVF-XL	BELGIUM
PF2290A	PVF-M	BELGIUM
PF2293A	PVF-M	BELGIUM
PF2390A	PVF-L	BELGIUM
PF2391A	PVF-L	BELGIUM



PF2392A	PVF-L	BELGIUM
PF2396A	PVF-XL	BELGIUM
PF2406A	PVF-L	BELGIUM
PF2434A	PVF-XL	BELGIUM
PF2463A	PVF-M	BELGIUM
PF2755A	PVF-L	BELGIUM
PF2872D	PVF-M	BELGIUM
PF2938A	PVF-M	BELGIUM
PF2946A	PVF-M	BELGIUM
PF2947A	PVF-M	BELGIUM
PF2961A	PVF-M	BELGIUM
PF2995A	PVF-XL	BELGIUM
PF3394A	PVF-XL	BELGIUM
PF3395A	PVF-XL	BELGIUM
PF3397A	PVF-XL	BELGIUM
PF3462A	PVF-XL	BELGIUM
PF3467A	PVF-XL	BELGIUM
PF3574A	PVF-S	BELGIUM
PE7081C	PVF-L	Bosnia-Herzegovina
PE7176C	PVF-L	Bosnia-Herzegovina
PF1165A	PVF-S	Bosnia-Herzegovina
PF1166A	PVF-S	Bosnia-Herzegovina
PF1661A	PVF-M	Bosnia-Herzegovina
PF2082C	PVF-S	Bosnia-Herzegovina
PE9586C	PVF-XL	BULGARIA
PE9587C	PVF-XL	BULGARIA
PE9810A	PVF-XL	BULGARIA
PE9816A	PVF-XL	BULGARIA
PF2241A	PVF-S	BULGARIA
PF2356A	PVF-S	BULGARIA
PF2358A	PVF-S	BULGARIA
PF2454A	PVF-S	BULGARIA
PF2475A	PVF-S	BULGARIA
PF2476A	PVF-S	BULGARIA
PF0478A	PVF-L	CHILE
PF0556A	PVF-L	CHILE
PF0559A	PVF-L	CHILE
PF0585A	PVF-XL	CHILE
PF0599A	PVF-XL	CHILE
PF0600A	PVF-XL	CHILE
PE6677C	PVF-L	FRANCE
PE6703C	PVF-XL	FRANCE
PE6850C	PVF-L	FRANCE
PE6851C	PVF-L	FRANCE
PE6853C	PVF-M	FRANCE
PE6854C	PVF-M	FRANCE
PE6868C	PVF-L	FRANCE



PE6874C	PVF-L	FRANCE
PE6885C	PVF-M	FRANCE
PE6909C	PVF-L	FRANCE
PE6954A	PVF-S	FRANCE
PE6964A	PVF-S	FRANCE
PE6966A	PVF-S	FRANCE
PE6969A	PVF-XL	FRANCE
PE6972A	PVF-L	FRANCE
PE6973A	PVF-S	FRANCE
PE7011A	PVF-S	FRANCE
PE7099A	PVF-S	FRANCE
PE7117A	PVF-L	FRANCE
PE7123C	PVF-L	FRANCE
PE7138C	PVF-XL	FRANCE
PE7183C	PVF-L	FRANCE
PE7186C	PVF-XL	FRANCE
PE7306A	PVF-XL	FRANCE
PE7338C	PVF-L	FRANCE
PE7408C	PVF-M	FRANCE
PE7470A	PVF-S	FRANCE
PE7471A	PVF-S	FRANCE
PE7518C	PVF-S	FRANCE
PE7519C	PVF-S	FRANCE
PE7681A	PVF-M	FRANCE
PE7682A	PVF-M	FRANCE
PE7683A	PVF-M	FRANCE
PE7020A	PVF-S	FRANCE
PE9578C	PVF-L	FRANCE
PE9814A	PVF-M	FRANCE
PE9817A	PVF-M	FRANCE
PF0090A	PVF-M	FRANCE
PF0383A	PVF-S	FRANCE
PF0427A	PVF-S	FRANCE
PF0453C	PVF-S	FRANCE
PF0460C	PVF-S	FRANCE
PF0667A	PVF-XL	FRANCE
PF1371A	PVF-XL	FRANCE
PF1912A	PVF-XL	FRANCE
PF1913A	PVF-XL	FRANCE
PF1922A	PVF-XL	FRANCE
PF2057A	PVF-XL	FRANCE
PF2160A	PVF-M	FRANCE
PF2161A	PVF-M	FRANCE
PF2179A	PVF-L	FRANCE
PF2188A	PVF-M	FRANCE
PF2230A	PVF-XL	FRANCE
PF2301A	PVF-M	FRANCE



PF2305A	PVF-M	FRANCE
PF2351A	PVF-M	FRANCE
PF2382A	PVF-L	FRANCE
PF2405A	PVF-M	FRANCE
PF2407A	PVF-M	FRANCE
PF2410A	PVF-L	FRANCE
PF2414A	PVF-L	FRANCE
PF2415A	PVF-L	FRANCE
PF2416A	PVF-XL	FRANCE
PF2438A	PVF-XL	FRANCE
PF2440A	PVF-XL	FRANCE
PF2448A	PVF-L	FRANCE
PF2470A	PVF-M	FRANCE
PF2603A	PVF-S	FRANCE
PF2612A	PVF-M	FRANCE
PF2614A	PVF-M	FRANCE
PF2860D	PVF-XL	FRANCE
PF2866D	PVF-XL	FRANCE
PF2868D	PVF-L	FRANCE
PF2873D	PVF-L	FRANCE
PF2888D	PVF-L	FRANCE
PF2907A	PVF-S	FRANCE
PF2981A	PVF-S	FRANCE
PF2993A	PVF-M	FRANCE
PF3006A	PVF-M	FRANCE
PF3007A	PVF-M	FRANCE
PF3054A	PVF-M	FRANCE
PF3140A	PVF-M	FRANCE
PF3229A	PVF-M	FRANCE
PF3354A	PVF-M	FRANCE
PF3377A	PVF-M	FRANCE
PF3383A	PVF-XL	FRANCE
PF3392A	PVF-XL	FRANCE
PF3443A	PVF-S	FRANCE
PF3509A	PVF-S	FRANCE
PF3523A	PVF-XL	FRANCE
PF3526A	PVF-S	FRANCE
PF3538A	PVF-S	FRANCE
PF3560A	PVF-S	FRANCE
PF3568A	PVF-XL	FRANCE
PF3593A	PVF-XL	FRANCE
PF3598A	PVF-XL	FRANCE
PE6663C	PVF-XL	GERMANY
PE6716C	PVF-XL	GERMANY
PE6717C	PVF-XL	GERMANY
PE6719C	PVF-XL	GERMANY
PE6827A	PVF-XL	GERMANY



PE6840C	PVF-M	GERMANY
PE6849C	PVF-L	GERMANY
PE6857C	PVF-L	GERMANY
PE6867C	PVF-L	GERMANY
PE6905C	PVF-M	GERMANY
PE6943A	PVF-L	GERMANY
PE6947A	PVF-L	GERMANY
PE6955A	PVF-S	GERMANY
PE6959A	PVF-L	GERMANY
PE6965A	PVF-S	GERMANY
PE6982A	PVF-L	GERMANY
PE7001A	PVF-L	GERMANY
PE7002A	PVF-L	GERMANY
PE7090A	PVF-L	GERMANY
PE7093A	PVF-M	GERMANY
PE7115A	PVF-M	GERMANY
PE7116A	PVF-M	GERMANY
PE7124C	PVF-L	GERMANY
PE7128C	PVF-XL	GERMANY
PE7131C	PVF-XL	GERMANY
PE7134C	PVF-L	GERMANY
PE7171C	PVF-L	GERMANY
PE7184C	PVF-L	GERMANY
PE7342C	PVF-L	GERMANY
PE7344C	PVF-XL	GERMANY
PE7432C	PVF-M	GERMANY
PE7466A	PVF-XL	GERMANY
PE7475A	PVF-XL	GERMANY
PE7509C	PVF-XL	GERMANY
PE7516C	PVF-S	GERMANY
PE7634A	PVF-S	GERMANY
PE7676A	PVF-S	GERMANY
PE7772A	PVF-XL	GERMANY
PE7870C	PVF-M	GERMANY
PE8563A	PVF-XL	GERMANY
PE9821A	PVF-M	GERMANY
PE9943A	PVF-S	GERMANY
PF0044A	PVF-S	GERMANY
PF0452C	PVF-L	GERMANY
PF0456C	PVF-L	GERMANY
PF0459C	PVF-S	GERMANY
PF0462C	PVF-S	GERMANY
PF0792A	PVF-XL	GERMANY
PF1658A	PVF-L	GERMANY
PF1897A	PVF-XL	GERMANY
PF1900A	PVF-XL	GERMANY
PF1925A	PVF-XL	GERMANY



PF2003A	PVF-XL	GERMANY
PF2021A	PVF-XL	GERMANY
PF2102C	PVF-XL	GERMANY
PF2152A	PVF-XL	GERMANY
PF2183A	PVF-M	GERMANY
PF2189A	PVF-XL	GERMANY
PF2199A	PVF-XL	GERMANY
PF2232A	PVF-XL	GERMANY
PF2237A	PVF-XL	GERMANY
PF2239A	PVF-XL	GERMANY
PF2285A	PVF-L	GERMANY
PF2309A	PVF-M	GERMANY
PF2384A	PVF-L	GERMANY
PF2408A	PVF-M	GERMANY
PF2418A	PVF-M	GERMANY
PF2435A	PVF-M	GERMANY
PF2452A	PVF-L	GERMANY
PF2464A	PVF-M	GERMANY
PF2474A	PVF-M	GERMANY
PF2608A	PVF-S	GERMANY
PF2639A	PVF-M	GERMANY
PF2828A	PVF-L	GERMANY
PF2870D	PVF-M	GERMANY
PF2894A	PVF-S	GERMANY
PF2974A	PVF-S	GERMANY
PF2980A	PVF-M	GERMANY
PF3009A	PVF-M	GERMANY
PF3072A	PVF-XL	GERMANY
PF3073A	PVF-XL	GERMANY
PF3078A	PVF-XL	GERMANY
PF3079A	PVF-XL	GERMANY
PF3255A	PVF-S	GERMANY
PF3344A	PVF-XL	GERMANY
PF3517A	PVF-S	GERMANY
PE6645A	PVF-L	GREECE
PE6658A	PVF-L	GREECE
PE6720C	PVF-M	GREECE
PE6768A	PVF-M	GREECE
PE7548A	PVF-S	GREECE
PE7562A	PVF-S	GREECE
PE7567A	PVF-S	GREECE
PE7586A	PVF-S	GREECE
PE7587A	PVF-S	GREECE
PE7601A	PVF-S	GREECE
PE7602A	PVF-S	GREECE
PE7796A	PVF-M	GREECE
PE7800A	PVF-M	GREECE



PE7865C	PVF-M	GREECE
PE9809A	PVF-L	GREECE
PE9813A	PVF-L	GREECE
PE9838A	PVF-L	GREECE
PE9842A	PVF-L	GREECE
PE9940A	PVF-L	GREECE
PE9957A	PVF-L	GREECE
PE9958A	PVF-L	GREECE
PF0093A	PVF-M	GREECE
PF0100A	PVF-M	GREECE
PF0101A	PVF-M	GREECE
PF0109A	PVF-M	GREECE
PF0110A	PVF-M	GREECE
PF1980A	PVF-XL	GREECE
PF2022A	PVF-XL	GREECE
PF2067A	PVF-XL	GREECE
PF2068A	PVF-XL	GREECE
PF2069A	PVF-XL	GREECE
PF2070A	PVF-XL	GREECE
PF2071A	PVF-XL	GREECE
PF2088A	PVF-XL	GREECE
PF2089A	PVF-XL	GREECE
PF2090C	PVF-XL	GREECE
PF2113C	PVF-XL	GREECE
PF2164A	PVF-S	GREECE
PF2211A	PVF-M	GREECE
PF2227A	PVF-M	GREECE
PF2229A	PVF-M	GREECE
PF2238A	PVF-M	GREECE
PF2264A	PVF-S	GREECE
PF2283A	PVF-S	GREECE
PF2319A	PVF-S	GREECE
PF2333A	PVF-M	GREECE
PF2345A	PVF-S	GREECE
PF2346A	PVF-M	GREECE
PF2357A	PVF-S	GREECE
PF2359A	PVF-M	GREECE
PF2360A	PVF-M	GREECE
PF2361A	PVF-S	GREECE
PF2375A	PVF-XL	GREECE
PF2377A	PVF-XL	GREECE
PF2379A	PVF-XL	GREECE
PF2383A	PVF-M	GREECE
PF2404A	PVF-M	GREECE
PF2447A	PVF-S	GREECE
PF2453A	PVF-S	GREECE
PF2455A	PVF-S	GREECE



PF2465A	PVF-S	GREECE
PF2485A	PVF-M	GREECE
PF3042C	PVF-XL	GREECE
PF0340A	PVF-S	HUNGARY
PF2194A	PVF-XL	HUNGARY
PF2292A	PVF-M	HUNGARY
PF2611A	PVF-L	HUNGARY
PF3172A	PVF-XL	HUNGARY
PE7339C	PVF-L	INDIA
PE7357C	PVF-L	INDIA
PE7363C	PVF-L	INDIA
PE7411C	PVF-L	INDIA
PE7760A	PVF-L	INDIA
PE7770A	PVF-L	INDIA
PE7771A	PVF-L	INDIA
PE7816A	PVF-L	INDIA
PE7852A	PVF-L	INDIA
PE7993C	PVF-L	INDIA
PE8000C	PVF-L	INDIA
PE8001C	PVF-L	INDIA
PE8002C	PVF-L	INDIA
PE8003C	PVF-L	INDIA
PE8010C	PVF-L	INDIA
PE8035C	PVF-L	INDIA
PE8036C	PVF-L	INDIA
PE8558A	PVF-L	INDIA
PE9947A	PVF-S	INDIA
PF0034A	PVF-S	INDIA
PF0104A	PVF-XL	INDIA
PF0107A	PVF-XL	INDIA
PF0116A	PVF-XL	INDIA
PF0174A	PVF-XL	INDIA
PF0369A	PVF-M	INDIA
PF0372A	PVF-M	INDIA
PF0384A	PVF-M	INDIA
PF0394A	PVF-M	INDIA
PF0405A	PVF-M	INDIA
PF0428A	PVF-M	INDIA
PF0513A	PVF-S	INDIA
PF0519A	PVF-S	INDIA
PF0533A	PVF-M	INDIA
PF0555A	PVF-M	INDIA
PF0557A	PVF-M	INDIA
PF2103C	PVF-XL	INDIA
PF2908A	PVF-S	INDIA
PF2919A	PVF-S	INDIA
PF2952A	PVF-S	INDIA



PF3008A	PVF-S	INDIA
PF3408A	PVF-M	INDIA
PF3413A	PVF-M	INDIA
PF3445A	PVF-M	INDIA
PF3446A	PVF-M	INDIA
PF3460A	PVF-M	INDIA
PF3484A	PVF-M	INDIA
PF3534A	PVF-M	INDIA
PF3550A	PVF-M	INDIA
PF3601A	PVF-M	INDIA
PF3605A	PVF-M	INDIA
PF3376A	PVF-XL	IRELAND
PF3381A	PVF-XL	IRELAND
PE7259C	PVF-M	ISRAEL
PE7369C	PVF-M	ISRAEL
PE7381C	PVF-M	ISRAEL
PE7382C	PVF-M	ISRAEL
PE7830A	PVF-M	ISRAEL
PE8561A	PVF-L	ISRAEL
PF0175A	PVF-XL	ISRAEL
PF0629A	PVF-XL	ISRAEL
PF0790A	PVF-S	ISRAEL
PF1872A	PVF-L	ISRAEL
PF1987A	PVF-S	ISRAEL
PF2083C	PVF-S	ISRAEL
PF2413A	PVF-L	ISRAEL
PF2785A	PVF-M	ISRAEL
PF3506A	PVF-XL	ISRAEL
PE6683C	PVF-L	ITALY
PE6684C	PVF-L	ITALY
PE6728C	PVF-L	ITALY
PE6772A	PVF-L	ITALY
PE6841C	PVF-M	ITALY
PE6852C	PVF-S	ITALY
PE6870C	PVF-M	ITALY
PE6872C	PVF-L	ITALY
PE6873C	PVF-M	ITALY
PE6875C	PVF-M	ITALY
PE6884C	PVF-M	ITALY
PE6892C	PVF-S	ITALY
PE6898C	PVF-M	ITALY
PE6914A	PVF-M	ITALY
PE6981A	PVF-L	ITALY
PE6983A	PVF-S	ITALY
PE7018A	PVF-M	ITALY
PE7047C	PVF-S	ITALY
PE7153C	PVF-XL	ITALY



PE7164C	PVF-XL	ITALY
PE7272C	PVF-L	ITALY
PE7337C	PVF-M	ITALY
PE7340C	PVF-L	ITALY
PE7341C	PVF-L	ITALY
PE7361C	PVF-XL	ITALY
PE7370C	PVF-M	ITALY
PE7371C	PVF-M	ITALY
PE7385C	PVF-M	ITALY
PE7460C	PVF-L	ITALY
PE7472A	PVF-S	ITALY
PE7484A	PVF-S	ITALY
PE7492A	PVF-S	ITALY
PE7502C	PVF-S	ITALY
PE7674A	PVF-L	ITALY
PE7677A	PVF-S	ITALY
PE7684A	PVF-XL	ITALY
PE9547C	PVF-S	ITALY
PE9785A	PVF-S	ITALY
PE9786A	PVF-S	ITALY
PF0091A	PVF-M	ITALY
PF0647A	PVF-XL	ITALY
PF1726A	PVF-S	ITALY
PF1739A	PVF-L	ITALY
PF1771A	PVF-L	ITALY
PF1777A	PVF-S	ITALY
PF1870A	PVF-S	ITALY
PF1898A	PVF-XL	ITALY
PF1978A	PVF-XL	ITALY
PF2216A	PVF-M	ITALY
PF2355A	PVF-M	ITALY
PF2419A	PVF-M	ITALY
PF2443A	PVF-L	ITALY
PF2467A	PVF-L	ITALY
PF2606A	PVF-M	ITALY
PF2783A	PVF-L	ITALY
PF2793A	PVF-XL	ITALY
PF2809A	PVF-M	ITALY
PF2829A	PVF-M	ITALY
PF2869D	PVF-M	ITALY
PF2879D	PVF-XL	ITALY
PF2880D	PVF-XL	ITALY
PF2903A	PVF-M	ITALY
PF2960A	PVF-M	ITALY
PF2976A	PVF-S	ITALY
PF2979A	PVF-M	ITALY
PF3005A	PVF-S	ITALY



PF3018C	PVF-S	ITALY
PF3160A	PVF-XL	ITALY
PF3216A	PVF-S	ITALY
PF3343A	PVF-XL	ITALY
PF3385A	PVF-M	ITALY
PF3409A	PVF-M	ITALY
PF3452A	PVF-S	ITALY
PF3458A	PVF-S	ITALY
PF3463A	PVF-S	ITALY
PE9822A	PVF-M	KOSOVO
PF2610A	PVF-L	LATVIA
PE6999A	PVF-L	LUXEMBOURG
PF2877D	PVF-XL	MONTENEGRO
PF2878D	PVF-XL	MONTENEGRO
PE6486A	PVF-XL	NETHERLANDS
PE6897C	PVF-L	NETHERLANDS
PE7054C	PVF-S	NETHERLANDS
PE7107A	PVF-M	NETHERLANDS
PE7109A	PVF-M	NETHERLANDS
PE7110A	PVF-M	NETHERLANDS
PE7129C	PVF-L	NETHERLANDS
PE9562C	PVF-S	NETHERLANDS
PF0652A	PVF-XL	NETHERLANDS
PF2373A	PVF-XL	NETHERLANDS
PF2437A	PVF-XL	NETHERLANDS
PF2886D	PVF-XL	NETHERLANDS
PF2898A	PVF-S	NETHERLANDS
PF2977A	PVF-M	NETHERLANDS
PF1914A	PVF-XL	NEW ZEALAND
PE7383C	PVF-M	NORTH MACEDONIA
PE7386C	PVF-M	NORTH MACEDONIA
PE8562A	PVF-L	NORTH MACEDONIA
PE8575A	PVF-M	NORTH MACEDONIA
PE9843A	PVF-L	NORTH MACEDONIA
PE9847A	PVF-L	NORTH MACEDONIA
PF1037A	PVF-M	NORTH MACEDONIA
PF1038A	PVF-M	NORTH MACEDONIA
PF1305A	PVF-L	NORTH MACEDONIA
PF1333A	PVF-L	NORTH MACEDONIA
PF1337A	PVF-S	NORTH MACEDONIA
PF1756A	PVF-M	NORTH MACEDONIA
PF1834A	PVF-S	NORTH MACEDONIA
PF2446A	PVF-S	NORTH MACEDONIA
PF3485A	PVF-XL	POLAND
PF3486A	PVF-XL	POLAND
PF3487A	PVF-XL	POLAND
PF3490A	PVF-XL	POLAND



PF3499A	PVF-XL	POLAND
PE6659C	PVF-S	PORTUGAL
PE6908C	PVF-L	PORTUGAL
PE7140C	PVF-M	PORTUGAL
PF2212A	PVF-M	PORTUGAL
PF3045C	PVF-S	PORTUGAL
PF3046C	PVF-S	PORTUGAL
PF3175A	PVF-S	PORTUGAL
PF3253A	PVF-S	PORTUGAL
PF3358A	PVF-XL	PORTUGAL
PF3406A	PVF-S	PORTUGAL
PF3564A	PVF-XL	PORTUGAL
PF3566A	PVF-XL	PORTUGAL
PF3501A	PVF-XL	ROMANIA
PF3505A	PVF-XL	ROMANIA
PF3596A	PVF-S	ROMANIA
PE7400C	PVF-M	SERBIA
PE7404C	PVF-M	SERBIA
PE7405C	PVF-M	SERBIA
PE7407C	PVF-M	SERBIA
PE7803A	PVF-XL	SERBIA
PE9772A	PVF-L	SERBIA
PE9774A	PVF-L	SERBIA
PE9585C	PVF-L	SERBIA
PF0432A	PVF-XL	SERBIA
PF0454C	PVF-S	SERBIA
PF0552A	PVF-XL	SERBIA
PF0560A	PVF-S	SERBIA
PF0562A	PVF-L	SERBIA
PF0582A	PVF-S	SERBIA
PF0601A	PVF-M	SERBIA
PF0623A	PVF-S	SERBIA
PF0649A	PVF-S	SERBIA
PF0654A	PVF-S	SERBIA
PF0692A	PVF-S	SERBIA
PF0994A	PVF-L	SERBIA
PF0995A	PVF-L	SERBIA
PF0996A	PVF-M	SERBIA
PF1039A	PVF-M	SERBIA
PF1040A	PVF-M	SERBIA
PF1160A	PVF-M	SERBIA
PF1244A	PVF-L	SERBIA
PF1246A	PVF-L	SERBIA
PF1247A	PVF-L	SERBIA
PF1255A	PVF-S	SERBIA
PF1304A	PVF-L	SERBIA
PF1331A	PVF-XL	SERBIA



PF1334A	PVF-S	SERBIA
PF1335A	PVF-S	SERBIA
PF1361A	PVF-XL	SERBIA
PF1567A	PVF-L	SERBIA
PF1583A	PVF-L	SERBIA
PF1641A	PVF-L	SERBIA
PF1642A	PVF-L	SERBIA
PF1650A	PVF-L	SERBIA
PF1701A	PVF-M	SERBIA
PF1742A	PVF-M	SERBIA
PF1743A	PVF-M	SERBIA
PF1796A	PVF-M	SERBIA
PF1802A	PVF-L	SERBIA
PF1831A	PVF-L	SERBIA
PF1832A	PVF-L	SERBIA
PF1842A	PVF-L	SERBIA
PF1843A	PVF-L	SERBIA
PF1992A	PVF-M	SERBIA
PF1993A	PVF-M	SERBIA
PF1994A	PVF-M	SERBIA
PF1995A	PVF-M	SERBIA
PF2056A	PVF-XL	SERBIA
PF2058A	PVF-XL	SERBIA
PF2176A	PVF-M	SERBIA
PF2185A	PVF-M	SERBIA
PF2231A	PVF-M	SERBIA
PF2300A	PVF-M	SERBIA
PF2348A	PVF-M	SERBIA
PF2349A	PVF-M	SERBIA
PF2363A	PVF-M	SERBIA
PF2411A	PVF-S	SERBIA
PF2607A	PVF-M	SERBIA
PF2861D	PVF-XL	SERBIA
PF2865D	PVF-XL	SERBIA
PF2901A	PVF-XL	SERBIA
PF2954A	PVF-XL	SERBIA
PF2975A	PVF-M	SERBIA
PF2978A	PVF-XL	SERBIA
PF2994A	PVF-XL	SERBIA
PF3191A	PVF-XL	SERBIA
PF3567A	PVF-XL	SERBIA
PF2192A	PVF-XL	SLOVAKIA
PF2193A	PVF-XL	SLOVAKIA
PF2471A	PVF-L	SLOVAKIA
PF2472A	PVF-L	SLOVAKIA
PE6906C	PVF-M	SOUTH KOREA
PE6936A	PVF-M	SOUTH KOREA



PE6960A	PVF-M	SOUTH KOREA
PE7009A	PVF-M	SOUTH KOREA
PE7010A	PVF-M	SOUTH KOREA
PE7012A	PVF-M	SOUTH KOREA
PE7017A	PVF-M	SOUTH KOREA
PE7019A	PVF-M	SOUTH KOREA
PE7037C	PVF-M	SOUTH KOREA
PE7043C	PVF-M	SOUTH KOREA
PE7044C	PVF-M	SOUTH KOREA
PE7071C	PVF-M	SOUTH KOREA
PE7126C	PVF-XL	SOUTH KOREA
PE7130C	PVF-M	SOUTH KOREA
PE7133C	PVF-M	SOUTH KOREA
PE7137C	PVF-M	SOUTH KOREA
PE7139C	PVF-M	SOUTH KOREA
PE7142C	PVF-M	SOUTH KOREA
PE7158C	PVF-M	SOUTH KOREA
PE7160C	PVF-S	SOUTH KOREA
PE7166C	PVF-M	SOUTH KOREA
PE7189C	PVF-S	SOUTH KOREA
PE7201C	PVF-M	SOUTH KOREA
PE7226C	PVF-M	SOUTH KOREA
PE7227C	PVF-M	SOUTH KOREA
PE7255A	PVF-S	SOUTH KOREA
PE7257A	PVF-S	SOUTH KOREA
PE7266C	PVF-S	SOUTH KOREA
PE7268C	PVF-L	SOUTH KOREA
PE7290C	PVF-M	SOUTH KOREA
PE7292C	PVF-M	SOUTH KOREA
PE7293C	PVF-M	SOUTH KOREA
PE7302A	PVF-M	SOUTH KOREA
PE7303A	PVF-M	SOUTH KOREA
PE7308A	PVF-L	SOUTH KOREA
PE7309A	PVF-L	SOUTH KOREA
PE7310A	PVF-M	SOUTH KOREA
PE7312A	PVF-S	SOUTH KOREA
PE7316A	PVF-L	SOUTH KOREA
PE7317A	PVF-L	SOUTH KOREA
PE7325A	PVF-M	SOUTH KOREA
PE7326A	PVF-M	SOUTH KOREA
PE7327A	PVF-M	SOUTH KOREA
PE7328A	PVF-M	SOUTH KOREA
PE7330A	PVF-S	SOUTH KOREA
PF0368A	PVF-S	SOUTH KOREA
PF0371A	PVF-S	SOUTH KOREA
PF1336A	PVF-L	SOUTH KOREA
PF1340A	PVF-S	SOUTH KOREA



PF1341A	PVF-S	SOUTH KOREA
PF1347A	PVF-S	SOUTH KOREA
PF1349A	PVF-S	SOUTH KOREA
PF1352A	PVF-S	SOUTH KOREA
PF1368A	PVF-L	SOUTH KOREA
PF1370A	PVF-M	SOUTH KOREA
PF1419A	PVF-S	SOUTH KOREA
PF1420A	PVF-L	SOUTH KOREA
PF1421A	PVF-L	SOUTH KOREA
PF1427A	PVF-S	SOUTH KOREA
PF1428A	PVF-XL	SOUTH KOREA
PF1430A	PVF-M	SOUTH KOREA
PF1431A	PVF-M	SOUTH KOREA
PF1432A	PVF-L	SOUTH KOREA
PF1433A	PVF-L	SOUTH KOREA
PF1434A	PVF-S	SOUTH KOREA
PF1435A	PVF-S	SOUTH KOREA
PF1436A	PVF-S	SOUTH KOREA
PF1438A	PVF-L	SOUTH KOREA
PF1440A	PVF-S	SOUTH KOREA
PF1441A	PVF-M	SOUTH KOREA
PF1447A	PVF-M	SOUTH KOREA
PF1448A	PVF-L	SOUTH KOREA
PF1450A	PVF-S	SOUTH KOREA
PF1451A	PVF-XL	SOUTH KOREA
PF1452A	PVF-XL	SOUTH KOREA
PF1454A	PVF-S	SOUTH KOREA
PF1458A	PVF-M	SOUTH KOREA
PF1465A	PVF-S	SOUTH KOREA
PF1481A	PVF-S	SOUTH KOREA
PF1483A	PVF-M	SOUTH KOREA
PF1496A	PVF-S	SOUTH KOREA
PF1499A	PVF-M	SOUTH KOREA
PF1521A	PVF-M	SOUTH KOREA
PF1522A	PVF-M	SOUTH KOREA
PF1523A	PVF-XL	SOUTH KOREA
PF1524A	PVF-XL	SOUTH KOREA
PF1525A	PVF-XL	SOUTH KOREA
PF1533A	PVF-S	SOUTH KOREA
PF1551A	PVF-S	SOUTH KOREA
PF1553A	PVF-XL	SOUTH KOREA
PF1554A	PVF-S	SOUTH KOREA
PF1557A	PVF-L	SOUTH KOREA
PF1568A	PVF-S	SOUTH KOREA
PF1573A	PVF-L	SOUTH KOREA
PF1576A	PVF-S	SOUTH KOREA
PF1584A	PVF-S	SOUTH KOREA



PF1585A	PVF-L	SOUTH KOREA
PF1588A	PVF-L	SOUTH KOREA
PF1590A	PVF-S	SOUTH KOREA
PF1599A	PVF-L	SOUTH KOREA
PF1600A	PVF-L	SOUTH KOREA
PF1601A	PVF-S	SOUTH KOREA
PF1602A	PVF-XL	SOUTH KOREA
PF1612A	PVF-XL	SOUTH KOREA
PF1613A	PVF-L	SOUTH KOREA
PF1614A	PVF-L	SOUTH KOREA
PF1648A	PVF-S	SOUTH KOREA
PF1656A	PVF-S	SOUTH KOREA
PF1657A	PVF-S	SOUTH KOREA
PF1659A	PVF-S	SOUTH KOREA
PF1662A	PVF-M	SOUTH KOREA
PF1664A	PVF-M	SOUTH KOREA
PF1684A	PVF-L	SOUTH KOREA
PF1762A	PVF-L	SOUTH KOREA
PF1779A	PVF-L	SOUTH KOREA
PF1784A	PVF-L	SOUTH KOREA
PF1801A	PVF-L	SOUTH KOREA
PF1805A	PVF-XL	SOUTH KOREA
PF1845A	PVF-L	SOUTH KOREA
PF1863A	PVF-S	SOUTH KOREA
PF1865A	PVF-L	SOUTH KOREA
PF1868A	PVF-S	SOUTH KOREA
PF2002A	PVF-M	SOUTH KOREA
PF2011A	PVF-M	SOUTH KOREA
PF2012A	PVF-M	SOUTH KOREA
PF2150A	PVF-M	SOUTH KOREA
PF2153A	PVF-L	SOUTH KOREA
PF2181A	PVF-L	SOUTH KOREA
PF2184A	PVF-L	SOUTH KOREA
PF2200A	PVF-L	SOUTH KOREA
PF2233A	PVF-L	SOUTH KOREA
PF2246A	PVF-L	SOUTH KOREA
PF2302A	PVF-M	SOUTH KOREA
PF2303A	PVF-M	SOUTH KOREA
PF2304A	PVF-M	SOUTH KOREA
PF2306A	PVF-M	SOUTH KOREA
PF2307A	PVF-M	SOUTH KOREA
PF2308A	PVF-M	SOUTH KOREA
PF2323A	PVF-M	SOUTH KOREA
PF2374A	PVF-XL	SOUTH KOREA
PF2376A	PVF-XL	SOUTH KOREA
PF2378A	PVF-XL	SOUTH KOREA
PF2397A	PVF-XL	SOUTH KOREA



PF2484A	PVF-L	SOUTH KOREA
PF2486A	PVF-L	SOUTH KOREA
PF2487A	PVF-L	SOUTH KOREA
PF3520A	PVF-XL	SOUTH KOREA
PF3521A	PVF-XL	SOUTH KOREA
PF3522A	PVF-XL	SOUTH KOREA
PF3537A	PVF-XL	SOUTH KOREA
PF3541A	PVF-XL	SOUTH KOREA
PE6642A	PVF-M	SPAIN
PE6644A	PVF-M	SPAIN
PE6660C	PVF-S	SPAIN
PE6774A	PVF-M	SPAIN
PE6843C	PVF-XL	SPAIN
PE6844C	PVF-L	SPAIN
PE6847C	PVF-L	SPAIN
PE6848C	PVF-L	SPAIN
PE6855C	PVF-XL	SPAIN
PE6860C	PVF-XL	SPAIN
PE6878C	PVF-L	SPAIN
PE6887C	PVF-L	SPAIN
PE6894C	PVF-XL	SPAIN
PE6895C	PVF-XL	SPAIN
PE6896C	PVF-L	SPAIN
PE6901C	PVF-XL	SPAIN
PE6903C	PVF-S	SPAIN
PE6911C	PVF-XL	SPAIN
PE6912A	PVF-S	SPAIN
PE6913A	PVF-S	SPAIN
PE6922A	PVF-M	SPAIN
PE6923A	PVF-M	SPAIN
PE6924A	PVF-XL	SPAIN
PE6925A	PVF-XL	SPAIN
PE6928A	PVF-XL	SPAIN
PE6929A	PVF-XL	SPAIN
PE6934A	PVF-M	SPAIN
PE6935A	PVF-M	SPAIN
PE6949A	PVF-S	SPAIN
PE6956A	PVF-XL	SPAIN
PE6967A	PVF-M	SPAIN
PE6968A	PVF-XL	SPAIN
PE6970A	PVF-M	SPAIN
PE6971A	PVF-L	SPAIN
PE6974A	PVF-S	SPAIN
PE6985A	PVF-L	SPAIN
PE6990A	PVF-XL	SPAIN
PE6991A	PVF-S	SPAIN
PE6992A	PVF-L	SPAIN



PE6994A	PVF-S	SPAIN
PE6998A	PVF-L	SPAIN
PE6957A	PVF-XL	SPAIN
PE7000A	PVF-XL	SPAIN
PE7016A	PVF-M	SPAIN
PE7023A	PVF-S	SPAIN
PE7025C	PVF-S	SPAIN
PE7028C	PVF-L	SPAIN
PE7040C	PVF-S	SPAIN
PE7041C	PVF-M	SPAIN
PE7042C	PVF-L	SPAIN
PE7087A	PVF-XL	SPAIN
PE7089A	PVF-XL	SPAIN
PE7092A	PVF-L	SPAIN
PE7094A	PVF-L	SPAIN
PE7096A	PVF-L	SPAIN
PE7100A	PVF-XL	SPAIN
PE7101A	PVF-L	SPAIN
PE7102A	PVF-S	SPAIN
PE7104A	PVF-XL	SPAIN
PE7108A	PVF-L	SPAIN
PE7118A	PVF-XL	SPAIN
PE7135C	PVF-S	SPAIN
PE7136C	PVF-S	SPAIN
PE7154C	PVF-XL	SPAIN
PE7159C	PVF-M	SPAIN
PE7165C	PVF-S	SPAIN
PE7319A	PVF-XL	SPAIN
PE7356C	PVF-L	SPAIN
PE7358C	PVF-S	SPAIN
PE7359C	PVF-L	SPAIN
PE7362C	PVF-L	SPAIN
PE7364C	PVF-S	SPAIN
PE7365C	PVF-S	SPAIN
PE7366C	PVF-S	SPAIN
PE7367C	PVF-S	SPAIN
PE7406C	PVF-L	SPAIN
PE7430C	PVF-S	SPAIN
PE7462A	PVF-XL	SPAIN
PE7463A	PVF-XL	SPAIN
PE7511C	PVF-XL	SPAIN
PE7564A	PVF-XL	SPAIN
PE7565A	PVF-XL	SPAIN
PE7722A	PVF-XL	SPAIN
PE7724A	PVF-XL	SPAIN
PE7725A	PVF-XL	SPAIN
PE7872C	PVF-M	SPAIN



PE8380C	PVF-XL	SPAIN
PE8434A	PVF-XL	SPAIN
PE8440A	PVF-M	SPAIN
PE8448A	PVF-L	SPAIN
PE8450A	PVF-XL	SPAIN
PE8536A	PVF-XL	SPAIN
PE8541A	PVF-M	SPAIN
PE8542A	PVF-L	SPAIN
PE8543A	PVF-L	SPAIN
PE8544A	PVF-L	SPAIN
PE8545A	PVF-L	SPAIN
PE8557A	PVF-M	SPAIN
PE8573A	PVF-M	SPAIN
PE9762A	PVF-M	SPAIN
PE9808A	PVF-M	SPAIN
PE9811A	PVF-M	SPAIN
PE9836A	PVF-XL	SPAIN
PE9949A	PVF-XL	SPAIN
PF0083A	PVF-L	SPAIN
PF0084A	PVF-L	SPAIN
PF0085A	PVF-L	SPAIN
PF0096A	PVF-XL	SPAIN
PF0191A	PVF-L	SPAIN
PF0265A	PVF-XL	SPAIN
PF0271A	PVF-XL	SPAIN
PF0289A	PVF-XL	SPAIN
PF0291A	PVF-XL	SPAIN
PF0319A	PVF-XL	SPAIN
PF0320A	PVF-XL	SPAIN
PF0323A	PVF-XL	SPAIN
PF0393A	PVF-S	SPAIN
PF0404A	PVF-S	SPAIN
PF0409A	PVF-XL	SPAIN
PF0430A	PVF-XL	SPAIN
PF0793A	PVF-XL	SPAIN
PF0797A	PVF-XL	SPAIN
PF1705A	PVF-L	SPAIN
PF1740A	PVF-L	SPAIN
PF1761A	PVF-L	SPAIN
PF1833A	PVF-S	SPAIN
PF1862A	PVF-S	SPAIN
PF1864A	PVF-S	SPAIN
PF1881A	PVF-S	SPAIN
PF1882A	PVF-S	SPAIN
PF2331A	PVF-XL	SPAIN
PF2350A	PVF-M	SPAIN
PF2364A	PVF-XL	SPAIN



PF2477A	PVF-M	SPAIN
PF2478A	PVF-M	SPAIN
PF2479A	PVF-M	SPAIN
PF2604A	PVF-S	SPAIN
PF2605A	PVF-S	SPAIN
PF2609A	PVF-S	SPAIN
PF2615A	PVF-S	SPAIN
PF2750A	PVF-S	SPAIN
PF2920A	PVF-S	SPAIN
PF2941A	PVF-S	SPAIN
PF2957A	PVF-S	SPAIN
PF2958A	PVF-XL	SPAIN
PF2959A	PVF-XL	SPAIN
PF3085A	PVF-XL	SPAIN
PF3086A	PVF-XL	SPAIN
PF3115A	PVF-XL	SPAIN
PF3116A	PVF-XL	SPAIN
PF3139A	PVF-XL	SPAIN
PF3157A	PVF-XL	SPAIN
PF3158A	PVF-XL	SPAIN
PF3159A	PVF-XL	SPAIN
PE9907A	PVF-M	SWITZERLAND
PE9911A	PVF-M	SWITZERLAND
PF2412A	PVF-XL	SWITZERLAND
PF1622A	PVF-M	TAIWAN
PF1660A	PVF-S	TAIWAN
PF1687A	PVF-M	TAIWAN
PF1688A	PVF-M	TAIWAN
PF1727A	PVF-S	TAIWAN
PF1774A	PVF-S	TAIWAN
PF1775A	PVF-S	TAIWAN
PF1825A	PVF-S	TAIWAN
PF1873A	PVF-S	TAIWAN
PF3504A	PVF-XL	TAIWAN
PF3558A	PVF-XL	TAIWAN
PE7181C	PVF-L	TURKEY
PE7236C	PVF-L	TURKEY
PE7387C	PVF-M	TURKEY
PE7388C	PVF-M	TURKEY
PE7389C	PVF-M	TURKEY
PE7390C	PVF-M	TURKEY
PE7397C	PVF-M	TURKEY
PE7398C	PVF-M	TURKEY
PE7399C	PVF-M	TURKEY
PE7412C	PVF-L	TURKEY
PE7757A	PVF-L	TURKEY
PE7794A	PVF-XL	TURKEY



PE7798A	PVF-XL	TURKEY
PE7799A	PVF-XL	TURKEY
PE7817A	PVF-XL	TURKEY
PE7826A	PVF-XL	TURKEY
PE7828A	PVF-XL	TURKEY
PE8044C	PVF-L	TURKEY
PE8046C	PVF-L	TURKEY
PE8051C	PVF-L	TURKEY
PE8053C	PVF-L	TURKEY
PE8055C	PVF-L	TURKEY
PF0317A	PVF-L	TURKEY
PF0318A	PVF-L	TURKEY
PF0322A	PVF-L	TURKEY
PF0324A	PVF-XL	TURKEY
PF0325A	PVF-XL	TURKEY
PF0328A	PVF-L	TURKEY
PF0332A	PVF-L	TURKEY
PF0333A	PVF-L	TURKEY
PF0335A	PVF-XL	TURKEY
PF0336A	PVF-XL	TURKEY
PF0337A	PVF-XL	TURKEY
PF0338A	PVF-XL	TURKEY
PF0341A	PVF-L	TURKEY
PF0342A	PVF-L	TURKEY
PF0370A	PVF-XL	TURKEY
PF0373A	PVF-XL	TURKEY
PF0395A	PVF-L	TURKEY
PF0396A	PVF-L	TURKEY
PF0397A	PVF-L	TURKEY
PF0410A	PVF-L	TURKEY
PF0429A	PVF-L	TURKEY
PF0457C	PVF-L	TURKEY
PF0520A	PVF-M	TURKEY
PF0521A	PVF-M	TURKEY
PF0553A	PVF-XL	TURKEY
PF0554A	PVF-S	TURKEY
PF0558A	PVF-S	TURKEY
PF0561A	PVF-M	TURKEY
PF0584A	PVF-M	TURKEY
PF0586A	PVF-M	TURKEY
PF0598A	PVF-L	TURKEY
PF0621A	PVF-L	TURKEY
PF0622A	PVF-XL	TURKEY
PF0630A	PVF-L	TURKEY
PF0632A	PVF-M	TURKEY
PF0634A	PVF-M	TURKEY
PF0635A	PVF-M	TURKEY



PF0645A	PVF-M	TURKEY
PF0648A	PVF-S	TURKEY
PF0653A	PVF-L	TURKEY
PF0666A	PVF-S	TURKEY
PF0668A	PVF-S	TURKEY
PF0690A	PVF-M	TURKEY
PF0696A	PVF-XL	TURKEY
PF0697A	PVF-XL	TURKEY
PF0698A	PVF-L	TURKEY
PF0699A	PVF-L	TURKEY
PF0702A	PVF-XL	TURKEY
PF0788A	PVF-L	TURKEY
PF0789A	PVF-S	TURKEY
PF0791A	PVF-L	TURKEY
PF0794A	PVF-M	TURKEY
PF0795A	PVF-M	TURKEY
PF0796A	PVF-M	TURKEY
PF1006A	PVF-M	TURKEY
PF1157A	PVF-S	TURKEY
PF1158A	PVF-L	TURKEY
PF1159A	PVF-S	TURKEY
PF1161A	PVF-L	TURKEY
PF1162A	PVF-L	TURKEY
PF1163A	PVF-S	TURKEY
PF1164A	PVF-S	TURKEY
PF1237A	PVF-L	TURKEY
PF1238A	PVF-L	TURKEY
PF1243A	PVF-L	TURKEY
PF1248A	PVF-L	TURKEY
PF1249A	PVF-L	TURKEY
PF1250A	PVF-L	TURKEY
PF1251A	PVF-L	TURKEY
PF1254A	PVF-S	TURKEY
PF1263A	PVF-L	TURKEY
PF1265A	PVF-L	TURKEY
PF1303A	PVF-M	TURKEY
PF1306A	PVF-S	TURKEY
PF1315A	PVF-S	TURKEY
PF1316A	PVF-M	TURKEY
PF1317A	PVF-S	TURKEY
PF1332A	PVF-XL	TURKEY
PF1342A	PVF-M	TURKEY
PF1346A	PVF-XL	TURKEY
PF1348A	PVF-L	TURKEY
PF1350A	PVF-M	TURKEY
PF1351A	PVF-XL	TURKEY
PF1353A	PVF-M	TURKEY



PF1354A	PVF-L	TURKEY
PF1355A	PVF-XL	TURKEY
PF1358A	PVF-XL	TURKEY
PF1359A	PVF-L	TURKEY
PF1362A	PVF-M	TURKEY
PF1369A	PVF-M	TURKEY
PF1437A	PVF-M	TURKEY
PF1446A	PVF-M	TURKEY
PF1453A	PVF-L	TURKEY
PF1459A	PVF-L	TURKEY
PF1479A	PVF-L	TURKEY
PF1480A	PVF-M	TURKEY
PF1482A	PVF-M	TURKEY
PF1497A	PVF-L	TURKEY
PF1498A	PVF-M	TURKEY
PF1501A	PVF-M	TURKEY
PF1502A	PVF-M	TURKEY
PF1534A	PVF-S	TURKEY
PF1536A	PVF-L	TURKEY
PF1545A	PVF-L	TURKEY
PF1546A	PVF-L	TURKEY
PF1552A	PVF-XL	TURKEY
PF1555A	PVF-L	TURKEY
PF1566A	PVF-S	TURKEY
PF1582A	PVF-S	TURKEY
PF1589A	PVF-L	TURKEY
PF1618A	PVF-M	TURKEY
PF1619A	PVF-M	TURKEY
PF1623A	PVF-M	TURKEY
PF1624A	PVF-M	TURKEY
PF1625A	PVF-M	TURKEY
PF1643A	PVF-M	TURKEY
PF1649A	PVF-M	TURKEY
PF1682A	PVF-S	TURKEY
PF1683A	PVF-S	TURKEY
PF1685A	PVF-L	TURKEY
PF1702A	PVF-M	TURKEY
PF1703A	PVF-S	TURKEY
PF1704A	PVF-S	TURKEY
PF1707A	PVF-L	TURKEY
PF1722A	PVF-M	TURKEY
PF1724A	PVF-S	TURKEY
PF1725A	PVF-L	TURKEY
PF1728A	PVF-S	TURKEY
PF1737A	PVF-L	TURKEY
PF1738A	PVF-L	TURKEY
PF1741A	PVF-M	TURKEY



PF1757A	PVF-M	TURKEY
PF1758A	PVF-M	TURKEY
PF1759A	PVF-S	TURKEY
PF1772A	PVF-L	TURKEY
PF1773A	PVF-L	TURKEY
PF1798A	PVF-M	TURKEY
PF1799A	PVF-M	TURKEY
PF1804A	PVF-M	TURKEY
PF1806A	PVF-XL	TURKEY
PF1807A	PVF-M	TURKEY
PF1808A	PVF-M	TURKEY
PF1809A	PVF-M	TURKEY
PF1822A	PVF-M	TURKEY
PF1823A	PVF-M	TURKEY
PF1824A	PVF-M	TURKEY
PF1826A	PVF-M	TURKEY
PF1827A	PVF-S	TURKEY
PF1828A	PVF-M	TURKEY
PF1829A	PVF-M	TURKEY
PF1830A	PVF-M	TURKEY
PF1835A	PVF-M	TURKEY
PF1836A	PVF-M	TURKEY
PF1837A	PVF-M	TURKEY
PF1838A	PVF-M	TURKEY
PF1841A	PVF-M	TURKEY
PF1844A	PVF-L	TURKEY
PF1846A	PVF-L	TURKEY
PF1867A	PVF-L	TURKEY
PF1871A	PVF-L	TURKEY
PF1892A	PVF-XL	TURKEY
PF1893A	PVF-XL	TURKEY
PF1901A	PVF-M	TURKEY
PF1902A	PVF-M	TURKEY
PF1903A	PVF-M	TURKEY
PF1926A	PVF-XL	TURKEY
PF1935A	PVF-M	TURKEY
PF1958A	PVF-M	TURKEY
PF1959A	PVF-XL	TURKEY
PF1960A	PVF-XL	TURKEY
PF1961A	PVF-XL	TURKEY
PF1988A	PVF-S	TURKEY
PF1989A	PVF-L	TURKEY
PF1990A	PVF-L	TURKEY
PF2004A	PVF-M	TURKEY
PF2005A	PVF-M	TURKEY
PF2006A	PVF-M	TURKEY
PF2007A	PVF-M	TURKEY



PF2010A	PVF-M	TURKEY
PF2075C	PVF-L	TURKEY
PF2076C	PVF-L	TURKEY
PF2077C	PVF-L	TURKEY
PF2081A	PVF-L	TURKEY
PF2085A	PVF-M	TURKEY
PF2086A	PVF-M	TURKEY
PF2087A	PVF-L	TURKEY
PF2100C	PVF-M	TURKEY
PF2101C	PVF-M	TURKEY
PF2110C	PVF-M	TURKEY
PF2151A	PVF-S	TURKEY
PF2162A	PVF-L	TURKEY
PF2177A	PVF-S	TURKEY
PF2178A	PVF-S	TURKEY
PF2182A	PVF-S	TURKEY
PF2187A	PVF-S	TURKEY
PF2201A	PVF-L	TURKEY
PF2213A	PVF-L	TURKEY
PF2217A	PVF-L	TURKEY
PF2226A	PVF-L	TURKEY
PF2228A	PVF-L	TURKEY
PF2240A	PVF-S	TURKEY
PF2262A	PVF-S	TURKEY
PF2263A	PVF-S	TURKEY
PF2265A	PVF-S	TURKEY
PF2266A	PVF-S	TURKEY
PF2267A	PVF-S	TURKEY
PF2268A	PVF-S	TURKEY
PF2284A	PVF-S	TURKEY
PF2286A	PVF-L	TURKEY
PF2287A	PVF-L	TURKEY
PF2288A	PVF-M	TURKEY
PF2294A	PVF-M	TURKEY
PF2310A	PVF-S	TURKEY
PF2311A	PVF-S	TURKEY
PF2320A	PVF-S	TURKEY
PF2324A	PVF-M	TURKEY
PF2325A	PVF-M	TURKEY
PF2326A	PVF-M	TURKEY
PF2329A	PVF-M	TURKEY
PF2330A	PVF-M	TURKEY
PF2335A	PVF-S	TURKEY
PF2336A	PVF-S	TURKEY
PF2338A	PVF-M	TURKEY
PF2342A	PVF-M	TURKEY
PF2343A	PVF-S	TURKEY



PF2344A	PVF-S	TURKEY
PF2347A	PVF-M	TURKEY
PF2473A	PVF-L	TURKEY
PF2483A	PVF-L	TURKEY
PF2687A	PVF-L	TURKEY
PF2867D	PVF-L	TURKEY
PF2874D	PVF-L	TURKEY
PF2875D	PVF-L	TURKEY
PF2876D	PVF-L	TURKEY
PF2892D	PVF-XL	TURKEY
PF2893D	PVF-L	TURKEY
PF2895A	PVF-XL	TURKEY
PF2896A	PVF-XL	TURKEY
PF2897A	PVF-XL	TURKEY
PF2899A	PVF-L	TURKEY
PF2913A	PVF-L	TURKEY
PF2914A	PVF-L	TURKEY
PF2939A	PVF-L	TURKEY
PF2940A	PVF-L	TURKEY
PF2991A	PVF-L	TURKEY
PF2992A	PVF-L	TURKEY
PF3011A	PVF-S	TURKEY
PF3038C	PVF-M	TURKEY
PF3039C	PVF-M	TURKEY
PF3040C	PVF-M	TURKEY
PF3041C	PVF-M	TURKEY
PF3047C	PVF-S	TURKEY
PF3074A	PVF-S	TURKEY
PF3147A	PVF-S	TURKEY
PF3148A	PVF-S	TURKEY
PF3150A	PVF-S	TURKEY
PF3161A	PVF-XL	TURKEY
PF3163A	PVF-XL	TURKEY
PF3164A	PVF-XL	TURKEY
PF3165A	PVF-XL	TURKEY
PF3169A	PVF-XL	TURKEY
PF3170A	PVF-XL	TURKEY
PF3171A	PVF-XL	TURKEY
PF3173A	PVF-S	TURKEY
PF3177A	PVF-M	TURKEY
PF3179A	PVF-S	TURKEY
PF3181A	PVF-M	TURKEY
PF3189A	PVF-XL	TURKEY
PF3190A	PVF-XL	TURKEY
PF3193A	PVF-XL	TURKEY
PF3194A	PVF-XL	TURKEY
PF3195A	PVF-XL	TURKEY



PF3196A	PVF-XL	TURKEY
PF3197A	PVF-XL	TURKEY
PF3198A	PVF-M	TURKEY
PF3202A	PVF-M	TURKEY
PF3211A	PVF-M	TURKEY
PF3212A	PVF-M	TURKEY
PF3217A	PVF-S	TURKEY
PF3219A	PVF-M	TURKEY
PF3232A	PVF-XL	TURKEY
PF3233A	PVF-XL	TURKEY
PF3234A	PVF-XL	TURKEY
PF3235A	PVF-XL	TURKEY
PF3236A	PVF-XL	TURKEY
PF3239A	PVF-XL	TURKEY
PF3240A	PVF-XL	TURKEY
PF3242A	PVF-XL	TURKEY
PF3243A	PVF-XL	TURKEY
PF3256A	PVF-M	TURKEY
PF3269A	PVF-XL	TURKEY
PF3272A	PVF-XL	TURKEY
PF3273A	PVF-XL	TURKEY
PF3274A	PVF-XL	TURKEY
PF3275A	PVF-XL	TURKEY
PF3461A	PVF-XL	TURKEY
PF3599A	PVF-XL	TURKEY
PE6665C	PVF-XL	UK
PE6676C	PVF-XL	UK
PE6842C	PVF-M	UK
PE6845C	PVF-M	UK
PE6846C	PVF-M	UK
PE7070C	PVF-M	UK
PE7075C	PVF-M	UK
PE7313A	PVF-XL	UK
PE7467A	PVF-S	UK
PE7503C	PVF-S	UK
PE7504C	PVF-M	UK
PE7529C	PVF-M	UK
PE7530C	PVF-L	UK
PE7626C	PVF-M	UK
PE7679A	PVF-S	UK
PE7732A	PVF-M	UK
PE7736A	PVF-M	UK
PE9846A	PVF-S	UK
PF0455C	PVF-S	UK
PF0458C	PVF-S	UK
PF1706A	PVF-L	UK
PF1797A	PVF-M	UK



PF2112C	PVF-XL	UK
PF2170A	PVF-XL	UK
PF2352A	PVF-M	UK
PF2365A	PVF-L	UK
PF2441A	PVF-L	UK
PF2442A	PVF-L	UK
PF2450A	PVF-M	UK
PF2468A	PVF-M	UK
PF2808A	PVF-M	UK
PF2830A	PVF-M	UK
PF2831A	PVF-M	UK
PF2953A	PVF-M	UK
PF3010A	PVF-S	UK
PF3032C	PVF-S	UK
PF3037C	PVF-S	UK
PF3057A	PVF-L	UK
PF3083A	PVF-L	UK
PF3444A	PVF-S	UK
PF3570A	PVF-XL	UK
PF3576A	PVF-M	UK
PF3600A	PVF-S	UK



